

Woohoo: It's you and einhorn!

Good sex, as you know (or are about to experience!), means being able to trust each other in the right moment and to feel safe. Here's how the two (or three!) of you can be safe and sexy with einhorn condoms.

#makemagichappen



The condom should be put on before any sexual contact. That's the only way to protect yourself from sexually transmitted diseases and unwanted pregnancy. And, you know, properly enjoy the next few minutes or hours (whoa there!).

Start magic

① Please be very careful when removing the condom from the packaging. Fingernails, jewellery, or hedgehogs don't belong anywhere near this sensitive sheath. Condoms are robust and resilient, but not indestructible.

② Pull the foreskin back on the erect penis and place the condom right side up on the tip.

③ Pinch the reservoir with your thumb and index finger to keep air out and roll it all the way down without tugging. Did you put it on inside out? Grab another one - there's already bodily fluid on the outside.

④ Ouu... done? Pull the still erect penis out while holding the condom in place to avoid the leakage of sperm.

⑤ Tie it up and wrap it in a paper towel. Don't try to flush it down the toilet, just toss it in the trash can!

Magic interrupted

Is it jamming or slipping? Check to see if the condom is fitting properly or put on a new one if necessary.

Ready for the encore?


 Go ahead - but with a new condom! Please only use each condom once.

Already wet:

During intercourse, the vagina should be wet, otherwise penetration can be seriously painful. It's not good for the condom either.

Lubrication is always welcome and is fun - try it out. But make sure you only use lubrication made for condom use! Oil based substances, like butter (huh?),

Vaseline, body oil, or other body lotion can make the condom porous.

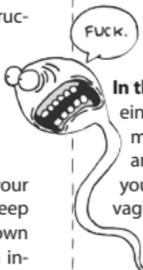

In the mood for some anal sex?

einhorn has got you covered. But remember: always use lubrication. The anus does not naturally lubricate. If you want to switch between anal and vaginal sex, make sure you don't use the same condom. The intestinal bacteria can lead to inflammation inside the vagina. So make sure you've got several condoms with you.

Oh no!

Did the condom slip off, rip or „burst?“ There's only one thing to do: visit the doctor as soon as possible within 72 hours. Find more on: www.einhorn.my/OhNo

BTW:

 **LATEX** Latex condoms are made from natural rubber and can be damaged by direct contact with medicinal products like oint-

ments. Make sure you ask your pharmacist or doctor before using them. Some people are also allergic to latex, which can lead to skin irritation. Even if rare, this can lead to anaphylactic shock. Use latex-free condoms if you are allergic.

Einhorn condoms offer effective protection against pregnancy and sexually transmitted infections or viruses. But just like all other contraception methods, 100% safety can't be guaranteed.

 Unicorns are notoriously sensitive creatures, just like einhorn condoms. Protect them from extreme heat and sunlight as well as moisture.

Safe and sound?

Does the individual container look damaged or does the expiration date look a bit ancient? Use another condom from a new package. Always check the expiration date before you stick one in your pocket.

Sexually transmitted infections:

Those who haven't had one (congrats) or believe so, often don't know what kind of infections there are. It's definitely worth knowing a bit more about this stuff, such as the symptoms of Chlamydia.

www.einhorn.my/lieblich

einhorn's shape

 einhorn condoms have a special flared shape: more intensity due to a roomier tip.



condoms

Woohoo! Ihr habt sie aufgerissen, die Tüte mit den einhorn Kondomen. Ihr wollt es also wissen. Sex ist spannend, macht glücklich, ist komisch, geil, hautnah, haut uns um. Wir fühlen uns gut. Guter Sex, ihr wisst es oder habt es noch vor euch, bedeutet einander im richtigen Moment vertrauen zu können und sich sicher zu fühlen. Mit einhorn Kondomen könnt ihr euch sicher sein. Jetzt geht es nur noch um euch:

#MAKEMAGICHAPPEN

